**MOPEDIA**

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Rialzo per water 12 cm

Rialzo per water 13 cm con braccioli

---

## **INSTRUCTION MANUAL**

*12 cm Raised toilet seat*

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Made in P.R.C.

## I Class Medical Device according to ECC directive 93/42

### CODES

**RP418** RAISED TOILET SEAT 12CM

**RP420** RAISED TOILET SEAT 13 CM WITH ARMRESTS

### INTENDED USE

Thank you for purchasing raised toilet seats MOPEDIA by Moretti S.p.A. manufactured to support people with movement difficulty. Their design and their quality guarantee maximum comfort, liability and safety. MOPEDIA by Moretti S.p.A. raised toilet seats are designed and realised to satisfy all your needs.

This instruction manual contains some useful tips for a correct use of the device you have bought as well as some important advises for your safety. In case of doubts, please contact your retailer who will be able to help you correctly.

### INTENDED USE

The MOPEDIA raised toilet seat is a device to help people with movement difficulties using the wc. Moretti S.p.A. declines all responsibilities for damages coming from an improper use of the device or from a use different from the one described in the present manual.

### EC DECLARATION OF CONFORMITY


Moretti S.p.A. declares under its sole responsibility that the products made and traded by Moretti S.p.A. and belonging to raised toilet seat range are in accordance with the European Directive on Medical devices (93/42/EEC).

Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The above mentioned products satisfy all the requirements as per annex I of the directive 93/42/CEE as per annex VII<sup>o</sup> of the before mentioned directive;
2. The above mentioned products are NOT MEASURING DEVICES
3. The above mentioned products are NOT INTENDED FOR USE IN CLINICAL VALIDATION;
4. The above mentioned products are NON STERILE;
5. The above mentioned products are to be considered "Class I<sup>o</sup> Medical Devices;
6. IT IS NOT ALLOWED to use the above mentioned products except for the use indicated by the Moretti S.p.A.;
7. The manufacturer makes all the technical documentations available to the relevant authorities which prove the compliance with the 93/42/EEC Directive;

### INSTRUCTIONS

1. Assembly armrests in the appropriate place of the raised toilet seat. Be sure they fit there correctly. **(pic.1) (only for model RP420)**
2. Be sure that the internal border is placed inside of toilet and the fore border is placed outside. Please note that the raised toilet seat has to fit together with pottery.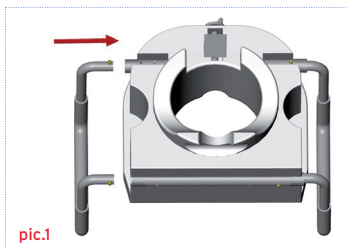


3. Turn the anterior handle clockwise until the MOPEDIA raised toilet seat is well fixed and can't be moved. (pic.2)

**WARNING**

The side-lock system has been especially designed to allow easy tightening therefore turning should not be continued once a resistance is felt.

**WARNING!**

Read carefully this instruction manual for a correct use. Pay attention to the presence of children.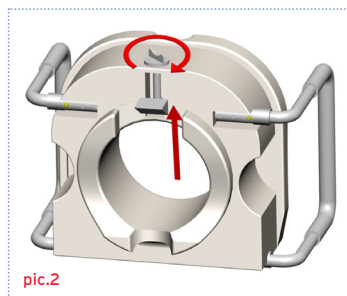

**CARE AND CLEANING**

All parts of the raised toilet seat can be cleaned with soap and warm water.

**GENERAL WARNINGS**

Do not use the product for a purpose not indicated in this manual. Moretti S.p.A. declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product. The manufacturer can modify the device or the instruction manual without prior notification.

**WARNING!**

Do not exceed the max loading weight of 110kg. Make sure the toilet seat is securely in place before putting weight on it.

**TECHNICAL FEATURES**

**RP418 Raised toilet seat**

- Max loading weight 110kg
- External dimensions 43x41,5x20 cm
- Internal border 27,5x23,5 cm
- Seat dimensions 43x41,5 cm
- Height of the raised toilet seat 12 cm
- Weight 2 Kg

**RP420 Raised toilet seat with armrests**

- Max loading weight 110kg
- External dimensions 52x43x385 cm with armrests
- Internal border 27,5x23,5 cm
- Armrest height from the seat 20 cm
- Seat dimensions 38x41 cm
- Height of the raised toilet seat 13 cm
- Weight 2,5 Kg

**WARRANTY**

The product has 2-YEAR warranty from purchase date which is certified by the receipt or invoice. The distributor reserves the right either to repair or replace necessary parts to correct defects in the materials or workmanship without any charge. The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alteration or disassembly by unauthorized individuals. In case you need to get the product repaired and you fulfil the conditions of warranty, send it to the nearest retailer together with the purchase receipt.

---

**WARRANTY CERTIFICATE**

Product \_\_\_\_\_

Purchasing date \_\_\_\_\_

Authorized dealer \_\_\_\_\_

Street \_\_\_\_\_ Place \_\_\_\_\_

Sold to \_\_\_\_\_

Street \_\_\_\_\_ Place \_\_\_\_\_

**MORETTI S.P.A.**INTERNET: [www.morettispacom](http://www.morettispacom) E-MAIL: [info@morettispacom](mailto:info@morettispacom)

Made in P.R.C.

**Moretti S.p.A.**

Via Bruxelles 3 - Meleto 52022 Cavriglia

( Arezzo )

Tel. +39 055 96 21 11

Fax. +39 055 96 21 200

[www.morettispa.com](http://www.morettispa.com)

[info@morettispa.com](mailto:info@morettispa.com)